



TAKE YOUR LIFE BACK!

PHONE: (719) 689-0203 FAX: (719) 689-5877
WEB: [HTTPS://OSTOMYARMOR.COM](https://ostomyarmor.com)



Brought to you by a Veteran.

Durable, Comfortable. Lasts for years: Guaranteed

Ostomy Armor Fit instructions. There is a video available on the website that shows how to measure.

When ordering, have ready the following information ready:

A. Waist/Torso size. B. Foam choice. C. Cut-out size. D. Color and Model. E. Buckle choice.

1. Waist measurement:

- a. The measurement is taken around the torso where the stoma is
- b. This may, or may not, be the same place as the customer's traditional "waist" measurement.
- c. The reason for this is that stomas are placed all over the abdomen. Some of them are in line with the actual waist, some are not. Some stomas are above, or below the belly button.
- d. People wear their pants in different places, i.e. high waisted and lower waisted
- e. Measure right under the stoma and pull the tape measure tight.
- f. If the customer thinks they will lose or gain a lot of weight, include that information.
 - i. E.G. after chemotherapy many people regain the weight they lost from ill health.
 - ii. Being more active (with the Armor) allows people to return to exercise.
 - iii. Some people gain a lot of weight from being inactive or from the effects of chemo.
- g. Ostomy Armor has about 2-3" of room to tighten and about 4-5" to loosen.

2. Foam pad sizes.

- a. Foam is available in Thin/.5", Standard/.75" or Deep/1"
- b. A stoma small in diameter that protrudes a small amount (less than .5") such as a urostomy, will only require the thin foam.
- c. Many people are very concerned that others will not know they have an ostomy and most people want a very slim profile. Under normal clothing (see website, home page at the bottom), it's hard to see the Armor. Under spandex/super tight clothing, the profile of the product will be seen.
- d. Most stomas that protrude up to one inch will work well with the Standard foam.
- e. Customers often choose the Deep foam if they take part in potentially high impact sports or work, such as ice hockey, or being a police officer. Many people choose Standard Foam for everyday activities, and another product in Deep foam for sports.
- f. Stomas that protrude more than about an inch will probably feel more comfortable with the Deep foam. The Deep foam will be more prominent under clothing.

3. Cut-out sizes. (the back of the cut-out shape in the foam.)

- a. Use the stoma size, not the barrier/baseplate size to determine size of cut-out
- b. Sizes offered are Standard/45mm, or Wide/50mm (measured at the widest point)

4. Buckle choice.

- a. Each model comes complete with a strong buckle as standard issue. There have never been any reports of buckle failure. Each model has the option of a Cobra Buckle for an additional cost. The Cobra buckles are super-strong and look fantastic.

5. Dual Armor:

- a. The same choices apply to the Dual Armor. Please contact us for sizing instructions. We will need digital photos.

Scott Aeromechanical R&D.

